

1 Sabax Road, Aeroton, Johannesburg, 2013, South Africa
P.o. Box 6888, Johannesburg, 2000, South Africa
Telephone: +27 11 494 8000
Email: info@adcock.com
GPS Co-ordinates: S 26 15 22.3 E 27 58 51.3

15 Jul 2026

www.adcock.com

MEDICAL DEVICE RECALL – CLASS III TYPE C

Dear Sir/ Madam,

Adcock Ingram Critical Care (AICC) is issuing a Medical Device Recall as the distributor for certain lots of HomeChoice Sets with 4-prong Cassettes due to an issue associated with an increase of reports of alarms while using the HomeChoice cycler. The most frequent alarms, “Check Lines & Bags” and “Reload the Set 156”, may occur and repeated alarms could prevent the HomeChoice system from initiating the therapy cycle.

The manufacturer, Vantive, has determined that this issue is caused only by specific cassettes from lots listed in the affected product table below:

Affected Product Table:

Product Code	Product Description	Lot Numbers
R5C4479	HOMECHOICE AUTOMATED PD SET WITH CASSETTE AND 4-PRONG LUERS	25L02T005 25L06T017 26A02T005

This issue could lead to a delay or interruption of therapy. In addition, it may potentially lead to touch contamination of the sterile fluid path due to failure of following aseptic technique as stipulated in the Patient Guide as a result of having to repeatedly change sets.

If any of the above batches have been received, please quarantine and isolate the impacted batch. Please contact the people below to arrange for upliftment and credit of the units.

Should you have any questions, please feel free to contact AICC.

Amina Mbonane (QA Release Manager): 011 494 8586
Nomazizi Semane (Senior Complaints Co-ordinator): 011 494 8744

Yours Sincerely,

Mr Ashley Jack
QA Manager (Authorised Rep): Adcock Ingram Critical Care Pty Ltd
0114948205
Ashley.jack@adcock.com

Adcock Ingram Critical Care (Pty) Ltd
Reg No 2000/004208/07
Directors: Mr A.G. Hall, Ms D. Neethling
Company Secretary: Mr M.A. Phalafala